Implantology

BoneFilter - BoneMill


Halle 10.2
Stand U 20

Schlumbohm

Titan KnochenFilter KF-T3

Titanium Bonefilter KF-T3

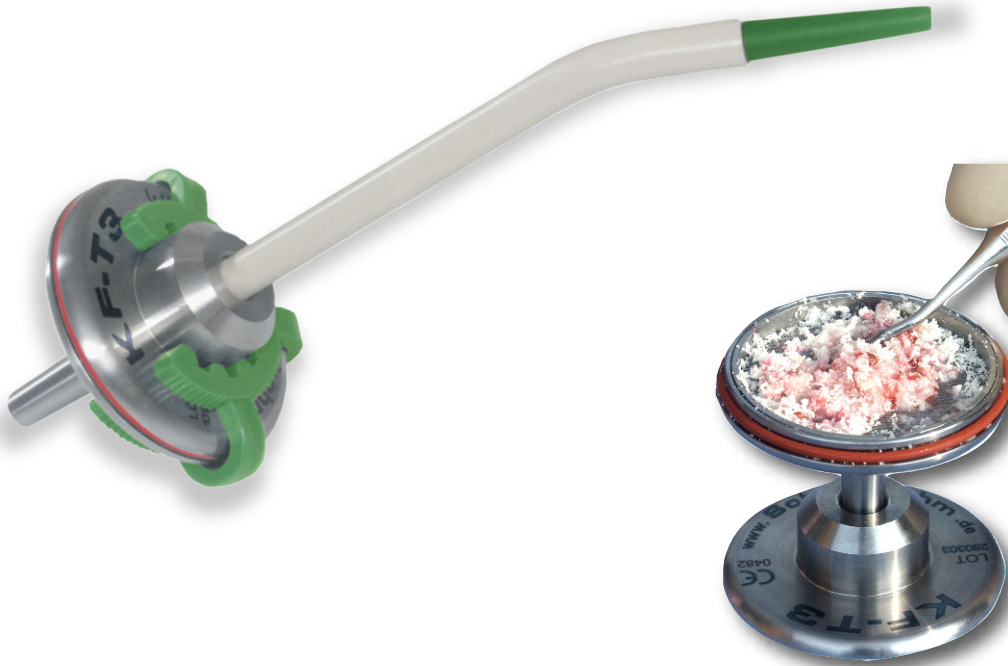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

With the modern augmentation, autologous bone proves on the basis of its osteoplastic potency and the lacking immunological reactions to be bone-substitute of first choice. The bone-quantity depends on the implant-size. It surpasses the implant-volume until approximately 2,4times.

In investigations of won bone material vital osteoblasts could be generated in the cell culture also without additive of osteogener factors. The emigration by osteoblasts was to be observed independently of the age of the patients or the donor region in comparable way.

(G.S.D. Chiriac, M.Herten, D.Rothamel u. Prof. J. Beckr, Heinrich-Heine Universität Düsseldorf 2004)

The advantages:

- timesaving extraction of autologous bone-material
- the especially big filter-surface guarantees maximum suction and interruption-free working
- easy handling – just plug and suck
- the titanium KF-T3 lies directly in the hand and guarantees high mobility during the treatment

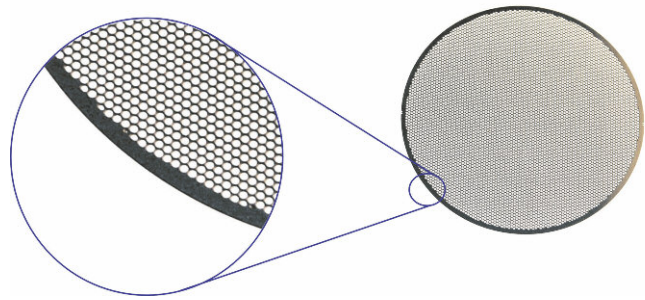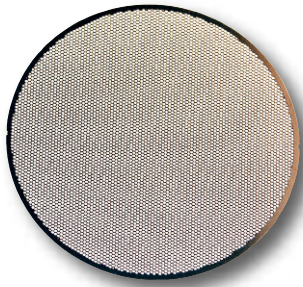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One-way-sieves

The new generation of one-way-sieves (MFS) has a surface-optimized geometry. The openings of the sieve has the form of honeycombs, that guarantees an utmost open surface. The smooth and flat surface of the MFS prevents a premature closing of the sievepores and eases the removal of the recovered bones chips.

104 3100	Titan-Knochenfilter T3 zweiteiliges Titan Gehäuse mit Klammer, Spannring sowie Silikon O-Ring	Titanium Bone Filter KF-T3 Titanium housing with clamp locking-ring and silikone O-ring	325,00* €
104 0370	Metallfolien-Einwegsiebe	One-way metalfoil-sieve	40,50* € 10 St./pcs
104 3380	Einweg-Sauger Surgitip Ø 2,5mm steril verpackt	Sterile surgical aspirator tip sterile	16,00* € 10 St./pcs
104 0380	Spannring	Locking ring	14,20* €
104 3150	Klammer KF-T3, grün	Clamp, (green)	12,80* €
104 3140	Silikon O-Ring KF-T3	Silicone O-ring	4,50* €
104 3385	Doppeladapter ø 11 /16 mm	Standard adapters ø 11 /16 mm	1,35* €

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*The prices listed are without obligation and are applicable for deliveries ex factory plus the applicable legal Value-Added-Tax. Prices are subject to change without notification; errors, omissions and misprints

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BoneMill KM-3

In order to correct big bone-defects, bone has to be taken out at other intra-oral positions frequently. The bigger bone-pieces, as well as bone-blocks, are out-separated with a trepan-drill for example. The bone-mill finds its use in grinding of these bone-blocks. The purpose is to generate an utmost volume at bone-chips from the bone-blocks. The chips should be granulated consistency to be well adaptable in the bone-bed. Autologues bone transplant has better properties than any other bone replacement material.

The special-cut-cylinder

The special-cut-cylinder of the bone-mill KM-3 has been developed in order to get granulated bone-chips. The cut-cylinder owns vacuum hardened straight, teethed with a special-wave-cut. Short granulated bone-pieces originate that show a length of 1,6 mm at most. These kind of bone-chips can be applied better so that the bone-growth of the new bone is secured. Studies confirm that autologous bone transplant is clearly superior to all allogenic, xenogenic and alloplastic materials for the reconstruction of bones. On the basis of the wave-cut, working the mill is easy.

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Schlumbohm GmbH & Co. KG